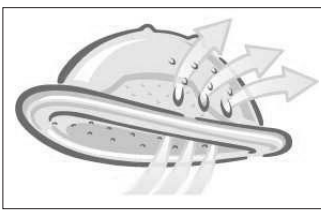
### D NUK Brustschalen-Set

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### GB NUK Set of Breast Shells

The NUK Set of Breast Shells (diagram A) has been designed to lend a helping hand during breastfeeding.  
Easy to use, the NUK Standard Breast Shells are designed to collect any breast milk leaking out during breastfeeding or when using a breast pump.  
The NUK Ventilated Breast Shells protect sensitive nipples from the discomfort of contact with clothing.  
The breast shells are anatomically shaped and the silicone cushions with gentle contours are comfortable to wear (Fig. A).

**Safety Instructions**  
When handling breast milk, hygiene is particularly important.  
Please observe and follow the instructions on this leaflet with regard to cleaning and sterilising. This will reduce any risk of contamination and help promote your child's healthy development.  
The NUK Set of Breast Shells is not a toy. This product is only to be used for its intended purpose, as described here.  
Retain contact details and Art.-no. for future reference in the event of a claim.

**Instructions for Standard Breast Shells**  
The Breast Shells are easily identified as the shells without ventilation holes (2).  
• During breastfeeding or when using a breast pump, milk can leak out of the "free" breast, as the milk flow is triggered in both breasts simultaneously.  
• The Breast Shell is applied to the free breast during breastfeeding or when using a breast pump.  
• The contours (4) on the silicone cushion (3) prevent suction occurring between cushion and breast.  
• For further information on how to proceed with the milk collected, please consult your health visitor, midwife, doctor or pharmacist.

**Instructions for Ventilated Breast Shells**  
The Ventilated Breast Shells are easily identified as the shells with 3 ventilation holes (2).  
• Breastfeeding mothers often experience sensitive or sore nipples. Any contact with the material of a nursing bra or breast pads can cause considerable discomfort. Using Ventilated Breast Shells prevents this contact, allowing the nipples time to recover.  
• This protection is designed to be worn in-between breastfeeding.  
• The shells (1) are anatomically shaped. The contours (4) on the silicone cushion (3) prevent suction occurring between cushion and breast.  
• The ventilation holes (2) ensure air-flow can always reach the nipples. The air circulation helps the nipples to recover.  
• Should milk escape, it should not be given to your baby under any circumstances.

**Instructions for use**  
During breastfeeding, use the breast shell without ventilation holes (2). In-between breastfeeding, use the ventilated breast shells with ventilation holes (2).  
1. Use only clean breast shells. For information on how to clean the shells, please refer to the section "Cleaning and Sterilising".  
**Important:** Before using for the first time, and after each use, the breast shells (1) and silicone cushions (3) must be thoroughly cleaned and sterilised.  
2. Wash hands thoroughly.  
3. Place the silicone cushion (3) over the breast shell (1).  
4. Place the breast shell in the nursing bra; the nipple should be in the centre of the opening (Fig. B).  
When using the ventilated breast shells, the ventilation holes must face upwards.  
5. After use, remove the shells from the nursing bra. The shells can be placed flat on a clean surface.  
6. Carefully remove the silicone cushion (3).  
7. Clean the breast shells as described in the following paragraph "Cleaning and Sterilising".

**Tipps:**  
• If milk leaks out of the shells during use, this can be soaked up with a paper tissue which will make cleaning easier afterwards. In order to do this, place the tissue on the underside of the breast shell (1). Please ensure that the ventilation holes (2) of the shells are not covered, thus allowing air to flow.  
• Tip to alleviate soreness: after breastfeeding, place a little breast milk onto the nipples and leave to dry naturally, or apply a natural lanolin balm.  
• If you have any questions, please consult your health visitor, midwife, doctor or pharmacist.

**Cleaning and Sterilising**  
**Important:** Before using for the first time, the breast shells (1) and silicone cushions (3) must be thoroughly cleaned and sterilised.  
1. After use, wash breast shells (1) and silicone cushions (3) thoroughly.  
2. Sterilise the breast shells and silicone cushions in the NUK Steam Sterilizer or by boiling them for at least 5 minutes.  
3. The cleaned NUK Set of Breast Shells can be safely stored by wrapping them in an ironed dish cloth or cotton diaper.

GREAT BRITAIN  
MAPA Spontex U.K. Ltd.  
Berkeley Business Park  
Wainwright Road  
Worcester WR4 9ZS  
☎ +44 (0) 1905-450328  
Fax: +44 (0) 1905-450340

IRELAND  
Pemberton International  
Unittech House Magna Business Park  
Magna Drive  
Citywest Road  
Dublin 24  
☎ +353 1 4598877  
Fax: +353 1 4041826

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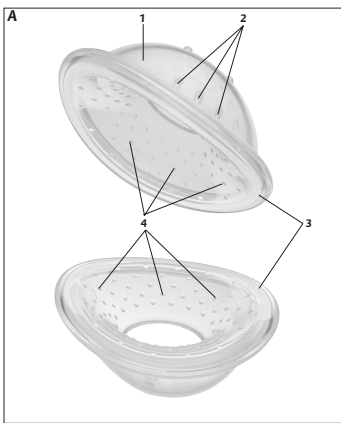
Please also bear in mind the other products from the NUK breast-feeding range, for example the NUK Nipple Shields  
During breastfeeding, NUK Nipple Shields protect and soothe sensitive nipples with soft, wafer-thin silicone. Soft nubbles gently stimulate the milk flow and thus help with breastfeeding. Due to the NUK Nipple Shield's unique cloverleaf shape, the child can maintain that all-important skin contact with its mother while breastfeeding.

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